

AccelStim™

Bone Healing Therapy



AccelStim Device Quick Instructions:

1. Connect the transducer to the AccelStim™ device by inserting the transducer cord into the transducer port.
2. Wear the strap such that the transducer will be on the area to be treated and secure it with the Velcro attached to the elastic strap.
3. Open the blue cover of the transducer holder by rotating it counterclockwise.
4. Apply the gel to the side of the transducer with no writing to form a 1-2 mm thick layer.
5. Insert the transducer inside the transducer holder so that the serial number is visible and then close the blue cover by rotating it.
6. Turn on the device by pressing On/Off Button for two seconds, until you hear a “beep”.
7. To start the treatment, press the Function Button of the device on the front panel.
8. Wear the device over treatment site for 20 minutes every day or as prescribed by your physician.
9. To recharge the battery, plug the power supply into the charging port.

Orthofix.com
BoneGrowthTherapy.com
1.800.535.4492

PN 20126754 REV AD 2022-03-24
AS-2103-US © Orthofix US LLC 3/2022

